

VAGINAL PATHOGENS by PCR

► **INTERMOUNTAIN CENTRAL LABORATORY** is offering a new Vaginal Panel by PCR **Wednesday, May 24th, 2023**. Current AFFIRM test code will only be available at Cassia, Idaho. Collection kits can be ordered by calling Lab Client Services at 801-507-2110

BATTERY CODE:	VAGPAN
COLLECT SPECIMENS:	Vaginal specimen (Clinician collected or clinician instructed self-collect)
COLLECTION CONTAINERS:	SWAB COLLECTION KIT MOLECULAR (ePro# 32072465)
PROCESSING INSTRUCTIONS:	Refrigerate upon collection
TRANSPORT:	Refrigerated
STABILITY:	Room temperature: 21 days Refrigerated: 21 days Frozen: Unacceptable
UNACCEPTABLE:	Samples collected using other swabs
PERFORMED/REPORTED:	Daily Reported next day

Key Points:

- **Vaginal Pathogen test code: VAGPAN**
- Test code go LIVE on 05/24/2023
- Must collect vaginal specimen using kit (ePro# 32072465) SWAB COLLECTION KIT MOLECULAR
- Do not use lubricants before collecting sample
- Avoid obscuring at least one full barcode on the top or bottom of the tube with the LIS label
- Update iCentra quick order favorites



SCAN THE QR CODE OR VISIT THE LINK BELOW TO
REVIEW THE TEST MENU ENTRY

<https://www.testmenu.com/Intermountain/Tests/1184942>


Intermountain[®]
Laboratory Services