



LMH LABORATORY:

Cervical/Vaginal Specimen Requirements and Instructions

URINE PCR testing

(GC/Chlamydia/Trich)

Male OR Female: Random Urine

(Trich NOT available for Men)

Collect Urine in Sterile Leak-Proof Container

OR

Fill to the Line on
Xpert Urine CTNG
Collection Device

Clean Catch/Cath Urine
are not acceptable

****DO NOT send small
Urine Culture Tube****



PCR testing on Cervical Swab

(GC/Chlamydia/Trich)

Female Only: Endocervical

Collect and place swab from Xpert CT/NG Vagi-
nal Endocervical Specimen Collection Kit into
PINK top transport reagent tube.


****Large swab for cleaning prior to collection,
then discard****



Group B Strep

Female Only: Endocervical/Rectal


Collect with Dual Swabs (one rectal other vagi-
nal) and place into Culturette

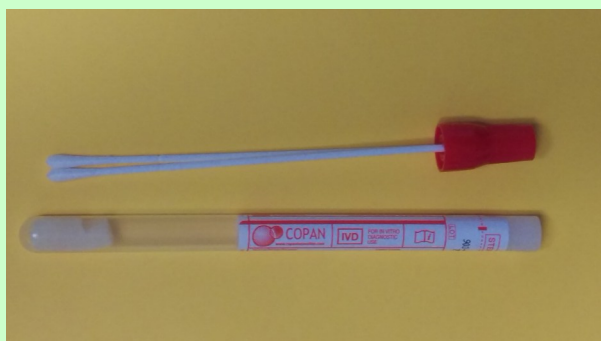

Cervical/Vaginal Culture

Female Only: Endocervical

Collect with Dual Swabs and place into Cul-
turette

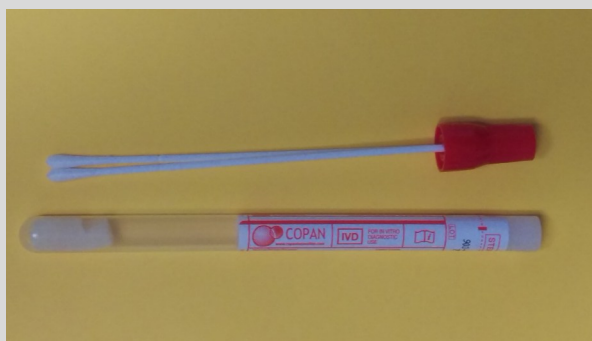

Wet Mount

Female Only: Endocervical

Collect using Cotton-tipped
Applicator swab. Place into white tube with
approx. 1 cc saline

