



## LMH LABORATORY:

### Cervical/Vaginal Specimen Requirements and Instructions

### URINE PCR testing (GC/Chlamydia/Trich)

**Male OR Female:** Random Urine

(Trich NOT available for Men)

Collect Urine in Sterile Leak-Proof Container

**OR**

Fill to the Line on  
Xpert Urine CTNG  
Collection Device

Clean Catch/Cath Urine  
are not acceptable

**\*\*DO NOT send small  
Urine Culture Tube\*\***



### PCR testing on Cervical Swab (GC/Chlamydia/Trich)

**Female Only:** Endocervical

Collect and place swab from Xpert CT/NG Vagi-  
nal Endocervical Specimen Collection Kit into  
**PINK** top transport reagent tube.


**\*\*Large swab for cleaning prior to collection,  
then discard\*\***



### Group B Strep

**Female Only:** Endocervical/Rectal

Collect with Dual Swabs (one rectal other vagi-  
nal) and place into Culturette

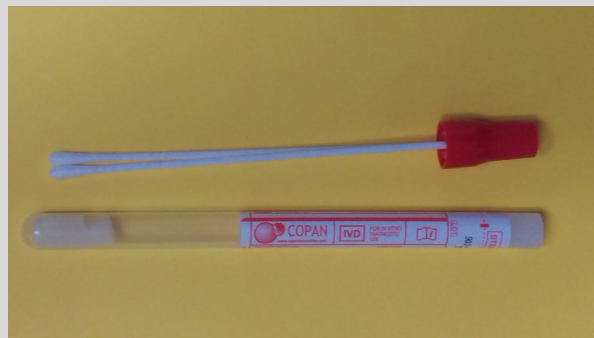

### Cervical/Vaginal Culture

**Female Only:** Endocervical

Collect with Dual Swabs and place into Cul-  
turette

