



LMH LABORATORY:

Genital/Cervical/Vaginal Specimen Requirements and Instructions

Last Updated: 11/2022

URINE PCR testing

(GC/Chlamydia/Trich)

Male OR Female: Random Urine

Collect Urine in Sterile Leak-Proof Container

OR

Fill to the Line on
Xpert Urine CTNG
Collection Device



Clean Catch/Cath Urine
are **NOT** acceptable

****DO NOT** send small Urine Culture Tube******

PCR testing on Cervical Swab

(GC/Chlamydia/Trich)

Female Only: Endocervical

Collect and place swab from Xpert CT/NG
Vaginal Endocervical Specimen Collection Kit
into **PINK** top transport reagent tube.

****Large swab for cleaning prior to collection,
then discard****



Group B Strep

Female Only: Endocervical/Rectal

Collect with Dual Swabs (one rectal, other
vaginal) and place into Culturette



Cervical/Vaginal Culture

Female Only: Endocervical

Collect with Dual Swabs and place into
Culturette

