


HOW TO USE BD™ ESwab and Universal Viral Transport

1

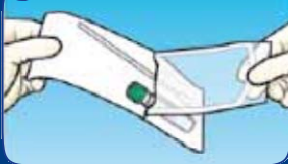

2



3




4



5



6

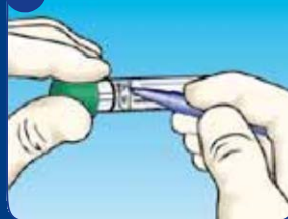

Microbiology Swabs

COLLECTION AND TRANSPORT GUIDE

BD ESwab Regular Collection Kit

White cap w/Liquid Amies and a regular flocked swab

Intended uses:

- Anaerobic/Aerobic Cultures
- Fungal Cultures
- AFB Cultures
- Group B Strep PCR
- MRSA Nasal PCR
- MRSA Wound PCR
- vanA VRE PCR
- Rapid Strep A
- Rotavirus (clean, dry container preferred)

BD ESwab Flexible Minitip Collection Kit

Blue cap w/Liquid Amies and flexible minitip flocked swab

Intended uses:

- Nasopharyngeal Culture
- Urethral Culture
- Bordetella pertussis/para PCR
- Legionella PCR
- Atypical pneumonia PCR /Chlamid)
- VZV PCR
- HSV 1-2 PCR
- CMV QUAL PCR

(When inserting, ensure that red line is below lip of tube prior to breaking)

Note: While the blue swab is acceptable for some PCR tests, the red swab is preferred.

BD Nasopharyngeal Swab / UVT Media Collection Kit

Red cap with flexible mini-tip flocked swab

Intended uses:

- Rapid RSV
- Flu A/B PCR and Flu A/B RSV PCR
- Respiratory Panel PCR
- Atypical pneumonia PCR (Mycopl/Chlamid)
- Bordetella pertussis/para PCR
- Legionella PCR
- VZV PCR
- HSV 1-2 PCR
- CMV QUAL PCR

Swab Collection Reminders:

- Have patient blow their nose (if nasal)
- Do not use lubricant (may use saline)
- Rotate 2-3 times
- Hold in place 5 seconds

For more information, please call (800) 324-7853.