

### ESwab Regular Collection Kit (Lawson #151477)

White cap w/Liquid Amies and a regular flocked swab

**Intended Uses:** 

All types of Anaerobic/Aerobic Cultures

Fungal Cultures MRSA Nasal PCR

AFB Cultures SA/MRSA PCR (Nasal Preop)

Group B Strep PCR MRSA Wound PCR Strep A PCR vanA VRE PCR

Mpox PCR Rotavirus \*clean dry container is preferred

C. auris Screen PCR



# Nasopharyngeal (NP) UTM/VTM Media Collection Kit (Lawson #189185) 3 mL\*

Red cap with universal/viral transport media and flexible mini-tip flocked swab

Intended Uses: Respiratory Panel PCR w/ COVID 19

COVID 19 Hospital PCR; COVID 19 BC PCR COVID 19-Flu A/B- RSV PCR, RSV/COVID PCR

Bordetella pertussis/para PCR

Legionella PCR

Atypical pneumonia PCR (Mycopl/Chlamid)

**VZV PCR and HSV 1-2 PCR** 

**CMV QUAL PCR** 

\*In case these tubes/swabs are not available, Nasopharyngeal (NP) swabs and UTM from other manufacturers are acceptable for all the above tests; swabs in Saline tubes are acceptable for all tests that include COVID.





## Nasal Swab UTM/VTM Media Collection Kit (Lawson #211973) 3 mL

Red cap with universal/viral transport media and flocked swab

Intended Uses\*: COVID 19 Hospital PCR

**COVID 19 BC PCR** 

\*COVID ONLY, See Power plans or standing orders



### **ESwab Flexible Mini-Tip Collection Kit**

(Lawson #156352)

Blue cap w/Liquid Amies and a flexible mini-tip flocked swab

Intended Uses: Nasopharyngeal Culture

**Urethral Culture** 

Bordetella pertussis/para PCR

Legionella PCR

Atypical pneumonia PCR (Mycopl/Chlam)

**VZV PCR and HSV 1-2 PCR** 

**CMV QUAL PCR** 

While the blue swab is acceptable for some PCR tests the red swab is the swab of choice



## Cepheid Xpert Swab Collection Kit-Lawson #169760

**Chlamydia-GC PCR (ED and Inpatients)** 

**Endocervical/Vaginal** 

Rectal

Pharyngeal (Throat)

**Trichomonas vaginalis PCR (ED and Inpatient)** 

**Endocervical/Vaginal** 

Vaginitis/Vaginosis Panel PCR

Vaginal swab

