







Test Name	Acceptable Collection	Information								
<p>HPV High Risk</p>	 <p>Thin Prep Lawson# 158323</p> <p>FLOQSwab for Self-Collection Lawson# 242100</p> <p>After collection, FLOQSwab® should be suspended in ThinPrep Pap Test PreservCyt® solution by trained personnel in clinical settings</p>	<p>HPV (Human Papilloma Virus) PCR test including HPV 16/18 Genotyping</p> <p><u>Co-testing</u> _____ 30-65 yrs of age</p> <p><u>ThinPrep PAP with HPV</u> _____ Negative Cytology / HPV positive for HPV 16 or 18 can be referred directly to colposcopy</p> <p><u>Reflex for ASCUS</u> _____</p> <p><u>ThinPrepPAP with Reflex HPV</u> ASCUS Cytology / HPV Positive (any High Risk HPV type) can be referred directly to colposcopy</p> <p>Our HPV test has Internal Control for sample adequacy.</p>								
<p>HSV 1+2 PCR VZV PCR</p>	<p>Red cap e swab/UVT</p> 	<p>Lesion Swabs:</p> <p>HSV (Herpes Simplex Virus (1 and 2) and VZV DNA detection in Muco-cutaneous Lesions</p>								
<p>Chlamydia/GC DNA (Outreach)</p> <p>Trichomonas vaginalis PCR (Outreach)</p> <p>Mycoplasma genitalium PCR (Outreach)</p>	<p>Roche Urine Kit Lawson# 176048</p>  <p>Thin Prep Lawson# 158323</p>  <p>Roche Swab Kit Lawson#212521</p> 	<table border="1"> <thead> <tr> <th data-bbox="873 1171 1235 1213">Specimen Type</th> <th data-bbox="1235 1171 1555 1213">Test</th> </tr> </thead> <tbody> <tr> <td data-bbox="873 1213 1235 1346">Urine (Male, Female), Genital, Vaginal, Endocervical</td> <td data-bbox="1235 1213 1555 1346">Chlamydia /GC Trichomonas vaginalis Mycoplasma genitalium</td> </tr> <tr> <td data-bbox="873 1346 1235 1451">ThinPrep</td> <td data-bbox="1235 1346 1555 1451">Chlamydia /GC, Trichomonas vaginalis</td> </tr> <tr> <td data-bbox="873 1451 1235 1535">Anal, Throat</td> <td data-bbox="1235 1451 1555 1535">Chlamydia /GC</td> </tr> </tbody> </table>	Specimen Type	Test	Urine (Male, Female), Genital, Vaginal, Endocervical	Chlamydia /GC Trichomonas vaginalis Mycoplasma genitalium	ThinPrep	Chlamydia /GC, Trichomonas vaginalis	Anal, Throat	Chlamydia /GC
Specimen Type	Test									
Urine (Male, Female), Genital, Vaginal, Endocervical	Chlamydia /GC Trichomonas vaginalis Mycoplasma genitalium									
ThinPrep	Chlamydia /GC, Trichomonas vaginalis									
Anal, Throat	Chlamydia /GC									
<p>Vaginitis/ Vaginosis Panel PCR</p>	<p>Xpert Swab Kit Lawson# 169760</p> 	<p>Vaginal Swabs ONLY:</p> <ul style="list-style-type: none"> • Bacterial vaginosis, • Candida group (<i>C. albicans</i>, <i>C. parapsilosis</i>, <i>C. tropicalis</i>, <i>C. dubliniensis</i>), • Candida glab-krus group (<i>C. glabrata</i>, <i>C. krusei</i>) • <i>Trichomonas vaginalis</i> 								