

Blood Lead Level Chart

Vein (Venous Draw)	Fingerstick or Heel stick (Capillary Draw)
<ul style="list-style-type: none"> Draw in royal blue tube Keep in royal blue tube Send to lab as whole blood DO NOT SPIN or separate 	<ul style="list-style-type: none"> Draw in microtainer lavender tube Keep in microtainer lavender tube Send to lab
<p>Consolidated Cerner Order: Lead Bld - ARUP</p>	<p>Consolidated Cerner Order: Lead Bld Cap – ARUP</p>
<p><u>NOTES/EXAMPLE:</u></p> <ul style="list-style-type: none"> Lead levels are done by type of drawn specimen either venous or capillary so follow the chart on what to send. Drawing lead levels is <u>not age dependent</u>, for example; if you draw a 2-year-old OR an 87-year-old by vein, then it must be sent in a royal blue tube and sent as whole blood. 	

REV: 08/2019
robin/homedrive/word/tests