

STD Swabs & Collection Devices

APTIMA Multitest SWAB Collection Kit

Sources of swab:

Rectal, Throat, Vaginal

Tests include:

- Chlamydia trachomatis (CT)
- Neisseria Gonorrhoeae (GC)
- Trichomonas vaginalis (Vaginal only)



APTIMA Unisex SWAB Collection Kit

Sources of swab:

Endocervical & Male Urethral

Tests include:

- Chlamydia trachomatis (CT)
- Neisseria Gonorrhoeae (GC)
- Trichomonas vaginalis (Endocervical only)



Submit blue-tip swab only to avoid rejection.

BD AFFIRM VP III SWAB

Used for Vaginal Panel by DNA probe. Includes:

- Candida species
- Gardnerella vaginalis
- Trichomonas vaginalis



cobas® PCR Dual SWAB Sample Packet

Used to detect presence of Chlamydia and/or Gonococcus (GC) in vaginal, rectal, throat, and Endocervical samples. Use **large swab** to collect **vaginal, throat, and rectal** samples. Use **small swab** to collect **Endocervical** samples.

Not to be used for medico-legal indications.



Cannot be used for Trichomonas

It is not necessary to collect this swab if ThinPrep Pap is obtained.

cobas® PCR URINE Sample Packet

Used to detect presence of Chlamydia and/or Gonococcus (GC) in urine samples. May be used for male or female collection. It is not necessary to collect this swab on females if ThinPrep Pap is obtained.

Not to be used for medico-legal indications.



First voided urine.
Must use this kit if specimen will not reach the lab within 24 hours.
Cannot be used for Trichomonas

ThinPrep Pap Vial

May be used in conjunction with Pap collection for

- Chlamydia trachomatis (CT)
- HPV High Risk
- Neisseria Gonorrhoeae (GC)
- Trichomonas vaginalis



**CONSOLIDATED
LABORATORY SERVICES**

St. Vincent's HealthCare

**For all your laboratory needs
904-308-5600**

October 27, 2020