

STOOL COLLECTION CHART

If the order is for

**Stool Culture & Sensitivity
Bacterial Enteric Pathogens**

included:

**Campylobacter
E. coli O157
Salmonella/Shigella
Yersinia**



Para-Pak C & S (Culture & Sensitivity)
ORANGE CAP Vial (as shown above)
Room temperature – 24 hours
Refrigerated – 72 hours

If the order is for

Stool for Parasites

Amoeba

Cryptosporidium Antigen

Giardia Antigen

Giardia (not a duplicate of Antigen)

Intestinal Parasites

Ova & Parasites

Worms



Para-Pak 10% Buffered Neutral Formalin
PINK CAP Vial (as shown above)
Room temperature – 24 hours
Refrigerated – 72 hours

If the order is for


***C. difficile* Toxin/Culture**

Fecal Lactoferrin

Rotavirus



Clean transport containers acceptable:
Clean transport container
Sterile urine cup
Refrigerate - DO NOT FREEZE

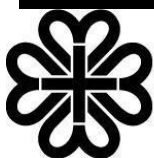

**CONSOLIDATED
LABORATORY SERVICES**

St. Vincent's HealthCare

For all your laboratory needs call

904-308-5600

REV: DEC2019