



Laboratory Common Test Collection Chart

Revision: 04/14/2020

Henepin County Medical Center

<p>X - Indicate preferred specimen container</p>	<p>Light Green Lithium Heparin</p>	<p>Dark Green Lithium Heparin, No Gel</p>	<p>SST</p>	<p>Red No Additive</p>	<p>Purple EDTA</p>	<p>Gray Sodium Fluoride/Potassium Oxalate</p>	<p>Blue Sodium Citrate</p>	<p>Minimum Volume Whole Blood</p>
Acetaminophen	X	X	X	X	X			1 mL
aPTT							X	Full Tube
Basic Metabolic, Electrolyte, Renal, Lipid, or Hepatic Panel	X	X	X	X				0.5 mL / panel
B-HCG	X	X	X	X				0.5 mL
B-hydroxybutyrate	X	X	X	X	X	X		0.5 mL
Bilirubin, Direct or Total	X	X	X	X				0.5 mL
Calcium	X	X	X	X				0.5 mL
CBC/Plt/Diff					X			1 mL
C-Reactive Protein	X	X	X	X				0.5 mL
Creatinine	X	X	X	X				0.5 mL
Electrophoresis, Hemoglobin					X			1 mL
Electrophoresis, Serum			X	X				1 mL
Ethanol	X	X	X	X	X	X		2 mL
Ethylene Glycol	X	X	X	X	X	X		2 mL
Ferritin	X	X	X	X				0.5 mL
Folate (Serum)			X	X				0.5 mL
Free Dilantin	X	X		X				4 mL
Glucose	X	X	X	X		X		0.5 mL
Glycosylated Hemoglobin (A1c)					X			0.5 mL
Hemoglobin					X			1 mL
Hep B Surface Antigen, Hep B Surface Antibody, or Hep C Antibody	X	X	X	X	X			0.5 mL / test
HIV Combo	X	X	X	X	X			0.5 mL
Immunodeficiency Panel (CD4)					X			1 mL
Ionized Calcium (ICA)	X-Ice	X-Ice						0.5 mL
Iron	X	X	X	X				0.5 mL
Lithium			X	X	X			0.5 mL
Magnesium	X	X	X	X				0.5 mL
NT-ProBNP	X	X	X	X				0.5 mL
Phosphorus	X	X	X	X				0.5 mL
Prottime & INR							X	Full Tube
Salicylate	X	X	X	X	X			1 mL

Total Protein	X	X	X	X				0.5 mL
Troponin I	X	X	X	X	X			0.5 mL
Urea Nitrogen-BUN	X	X	X	X				0.5 mL
Valproate	x	X		X				1 mL
Vancomycin		X		X	X			1 mL
Vitamin D 25-OH	X	X	X	X				1 mL

For collection information on additional tests, please refer to the electronic Laboratory Test Directory.