

BUH Laboratories —Urine Aliquot Guide by Department



White Top Vacuette—No Additive

R/MCHMA

Urine Protein Creatinine Ratio, random **UTPCR**

Urine Amylase, random **UAMY**

Urine Calcium, random, **UCAR**

Urine Chloride, random **UCLR**

Urine Potassium, random **UKR**

Urine Sodium, random **UNAR**

Urine Electrolytes, random **ULYR**

Urine Creatinine, random **UCRER**

Urine Microalbumin, random **UMICA**

Urine Osmolality, random **UOSMO**

Urine Protein, random **UTPR / OBTP**

Urine Urea Nitrogen random **UUNR**

Urine Uric Acid, random **UUAR**

Urine Phosphorus, random **UPHOR**

Urine Fentanyl **UFENT**

Urine Legionella **ULEGI**

Urine Alcohol Metabolites, random **ALCMT**

Urine Adulteration Screen (**MCHMA**) **RCUDS ADULT**

Urine Chemistry Drug Screen (**MCHMA**) **UAMP1, UBAR1, UBEN1, UCAN1, UCOC1, UMTD1, UOPI1, UPCP1, UBUPR, UOXYs, etc.**



Sterile, Screw Cap Collection Container

R/MTOX, R/MCLIM

Urine Toxicology Drug Screen (**RTOX**) **TDS, UN-BUP, RCUDS**

Urine Toxicology Confirmation (**RTOX**) **ACONF, BBCON, BUCON, COCON, MCONF, TCONF, BZCON, OPCON, etc**

Electrophoresis Orders (**R/MCLIM**)

Urine Protein Electrophoresis, Random **UPEP**

Urine immunofixation electrophoresis **UIFE**



BD Vacutainer—UA No Additive

R/MURNA

Urinalysis **UA / UMIC**

Urine Adulteration Screen (**MURNA**) **ADULT**

Urine Pregnancy **UHCG**

Urine Eosinophils **UEOS**

Urine Hemoglobin **UHGB**



BD Vacutainer—Urine C&S Preservative

Urine Culture CXURN

Please note: Do not open, or aliquot 12 hour or 24 hour urine collections; send in the original jug(s).