


GET IT STRAIGHT. REDUCE THE WAIT.

LABELS MUST BE INTACT AND READABLE

Logan Health Laboratory 752-1737 or x2396

PROPERLY LABELED

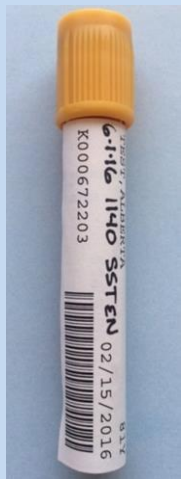


Front




Back

1. Place label directly under cap
2. Place label over manufacturer label
3. Only one label per tube
4. Make sure mnemonic or initials do not interfere with other patient information or lab barcodes.
5. Leave visible window to see blood.

UNACCEPTABLE

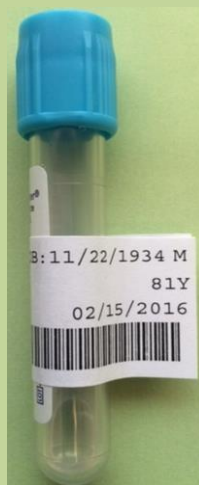

Flag

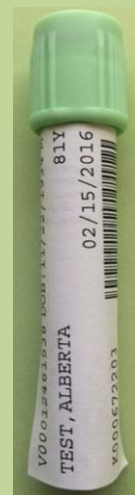


Upside down



Candy cane



Wraparound

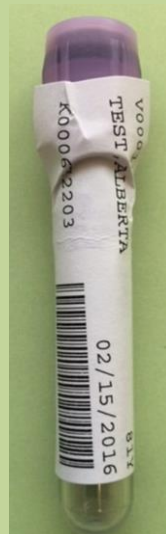

Scrunched



Bundled



Wrinkled

DOUBLE CHECK ALL LABELS BEFORE SENDING

- Full legal name (First and Last)
- Unique identifying number (DOB)
- Date & Time of collection
- Collector's mnemonic or initials

LOGAN
HEALTH