

CORRECT LABELING OF LABORATORY SPECIMENS

LIS Label

Leave space between stopper and label

Leave window for blood volume to be visible

HIS Label

Lab label should be placed on top of the manufacturer's label

Extra lab label wrapped on tube. Leave backing of label

DO NOT fold, pinch or tear lab label

BLOOD CULTURE LABELS

Do not write or cover the manufacture barcode

Do not place label on bottom of the container

BLOOD GAS LABELS

Correct way to label Blood Gas with LIS label

Correct way to label Blood Gas with HIS

PEDIATRIC LABELS

DO NOT fold, pinch or tear lab label

QUANTIFERON LABELS

Acceptable: Within lines OR On lines

Unacceptable: Underfilled

Unacceptable: Overfilled

Questions?

Call McLaren Medical Laboratory Customer Service at 810-396-5715