

UnityPoint-Meriter Laboratories
Information:
36 S. Brooks St.
Madison WI 53715

Patient
(chip or Lab label)

Patient Glucose Meter/Laboratory Correlation

Instructions for Clinic/Floor:

1. Order Epic **test 492073**.
2. **Patient should not have eaten, dosed or been given insulin within 2 hours of this test being performed.**
3. Inpatient orders will be drawn by Inpatient Phlebotomy in the patient's room.
4. All other patients should be sent to 2 Center Lab with an OB chart label for patient identification.

Instructions for Phlebotomy staff:

1. LIS will print two labels for "GLMTR" test. Phlebotomy staff will then fill out a Glucose Meter Correlation Sheet and attach one of the SOFT labels to the area in the upper right hand corner of the form.
2. Collect one Lithium Heparin plasma blood specimen.
3. Direct the patient to immediately (within 5 minutes of lab draw) perform a glucose test using their **personal** blood glucose meter.
4. Fill in the patient fingerstick glucose result from their **personal** blood glucose meter on the form while still with the patient.
5. Mark test as collected in the lab computer.
6. Send the Lithium Heparin tube along with this paperwork to the laboratory

Test Information: *(Phlebotomist staff- fill out)*

ML Order Number: _____ (or put on small sticker of lab label)

Collection Date: _____

Collection Time: _____

****Patients Meter Result:** _____