

UnityPoint-Meriter Laboratories
36 S. Brooks St.
Madison WI 53715

Patient Label

Patient Glucose Meter/Laboratory Correlation

Instructions for Clinic/Floor:

1. Order Epic test **LAB5061, Glucose, Meter Correlation.**
2. **Patient should not have eaten, dosed or been given insulin within 2 hours of this test being performed.**
3. Inpatient orders will be drawn by Inpatient Phlebotomy in the patient's room.
4. All other patients will have their collection completed within a Clinic Draw Station or the 2 Center Draw Station within the hospital.

Instructions for Phlebotomy staff:

1. LIS will print two labels for "GLMTR" test. Phlebotomy staff will then fill out a Glucose Meter Correlation Sheet and attach one of the Lab labels to the area in the upper right hand corner of the form.
2. Collect one Lithium Heparin plasma blood specimen.
3. Direct the patient to immediately (within 5 minutes of lab draw) perform a glucose test using their **personal** blood glucose meter.
4. Fill in the patient fingerstick glucose result from their **personal** blood glucose meter on the form, while still with the patient.
5. Send the Lithium Heparin tube along with this paperwork to the laboratory.

Test Information: (*Phlebotomist staff- fill out*)

ML Accession Number: _____(or Lab label)

Collection Date: _____

Collection Time: _____

****Patients Meter Result:** _____