



BD Urine Collection Device for Mid-Stream Collection

1) Gather Supplies:

For Urinalysis and Urine Culture use NEW Specimen Collection kit:
Cup, C&S Tube, UA Tube, Castile Wipe



For Urine Chemistry and other tests use:
Urine Chemistry Tube and Urine Transfer Device (available separately)

- Drug screens: AMPH, TCA, PCP
- Pregnancy
- UPEPs (send 2 chemistry tubes)
- Low volume collections (< 3mls for culture and/or < 7mls for UA)



2) Mid-Stream Collection Steps:



Step 1: Give patient a castile wipe and cup to void.



Step 2: Remove yellow sticker from lid.



Step 3: Collect urine specimen from sharp lid of cup.



Step 4: Invert urine tube 8-10x.

3) Discard urine, urine cup & lid:



Excess Urine



BD Urine Lid



Empty Urine Cup



Dirty sink or toilet



Large Sharps



Garbage

NEW!

Use the built in straw to collect the urine specimen.

Do NOT send urine cup to lab- it will be rejected!

Label specimens in the presence of the patient. Only send tubes to lab.

Refer to PROC 262: Specimen Collection