

BD Affirm™ VPIII Collection and Transport


BD Affirm™ VPIII Ambient Temperature Transport System

For transport of vaginal specimens for use in the BD Affirm™ VPIII Microbial Identification Test for *Candida*, *Gardnerella* and *Trichomonas*.

TRANSPORT




1

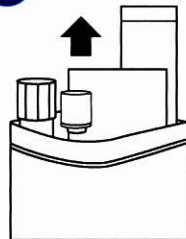


Open Seal / Remove Components

2

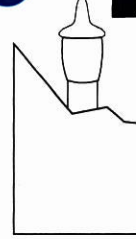

Tear / Remove Dropper

3

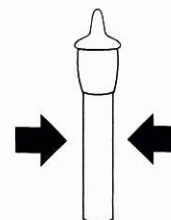


Break Ampule

4



Dispense into Tube

5

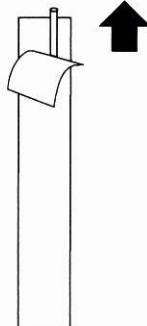


Peel / Remove Swab

6




Collect Sample from Patient

7

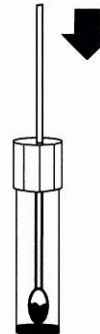

Place Swab in Tube

8

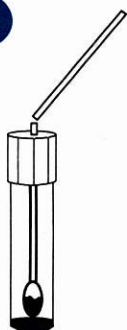

Break Shaft at Score Line

9



Close Cap Firmly

10



Place Patient Label on Tube



BD Diagnostics
7 Loveton Circle
Sparks, MD 21152-0999
800.638.8663
www.bd.com/ds

Sample is stable for 72 hours at ambient temperature (15-30°C) or refrigerated (2-8°C) storage.

BD, BD Logo and Affirm are trademarks of Becton, Dickinson and Company. ©2007 BD 1-222448 September 2007 Printed in USA