



Testing: Urine for Culture (AER, FUN, PPMU, VB3)

**Instructions:** Transfer 3cc of urine into a gray-capped tube using the straw provided

Urine Collection Kit (Transfer Straw, C&S tube)

## Collect Urine



## **Testing: Urine for Urinalysis**


**Instructions:** Transfer a minimum of 7mL of urine into marble top tube using the straw, invert tube 8-10



**Tube Urinalysis** 

## **Testing: Urine for GC/Chlam/Trich Amplification**

Instructions: Remove cap from yellow Aptima Urine tube (unscrew). Using the provided pipette, transfer urine to tube, ensuring urine is between fill lines

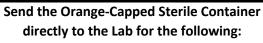

Urine Cytology Testing (50-100mL), UA
Testing, Mycoplasma/Ureaplasma Testing,
Leptospira PCR Testing, Low Volume Urine
Samples (< 3cc)





swab APTIMA URINE