

EMERGENCY RELEASE FOR CENTRIFUGE

Emergency Rotor Chamber Entry on Drucker 642e

In the event of **power failure**, it may be impossible to unlock the lid by conventional means. In this case, entry into the rotor chamber may be made by removing the latch label and using a pen to manually disengage the locking mechanism (see photo).

Pull the mechanism towards the control panel and then unlatch and open the lid. If the unit is damaged, please contact Sarah Bush Lincoln Health Center.



Peel back the right side of the label located at the latch. Reveal the opening underneath the label, insert a pen and pull the mechanism towards the front of the centrifuge.



SBL Laboratory Services

Sarah Bush Lincoln Laboratory
1000 Health Center Drive • Mattoon, IL 61938
1-800-573-1940

SBL Effingham Laboratory
905 North Maple • Effingham, IL 62401
1-877-796-LABS (5227)

testmenu.com/sarahbush