

# PROTECT FROM LIGHT SPECIMENS

In addition to temperature control, some analytes require protection from light to ensure specimen integrity.

Please collect the specimen tube with standard practice and label the specimen tube directly. **After collection**, protect the specimen from light by tightly wrapping the specimen in the ***preferable media, foil***. Alternatively, a paper towel, glove, or any other light protecting material can be used. The following tests require specimen light protection.

Test ID	Test Name
0090161	Amiodarone-ARUP
C0026	Bilirubin Profile
C0027	Bilirubin, Direct
C0028	Bilirubin, Neonatal Profile
C0029	Bilirubin, Total
0080055	Carotene-ARUP
0070385	Folate, RBC-ARUP
0081123	Vitamin B2 (Riboflavin)-ARUP
0080111	Vitamin B6 (Pyridoxal 5-Phosphate)-ARUP
0080380	Vitamin C (Ascorbic Acid) Plasma-ARUP
*0099225	Vitamin K1 Serum-ARUP (Label and cover specimen prior to collection)*



## SBL Laboratory Services

Sarah Bush Lincoln Laboratory  
1000 Health Center Drive • Mattoon, IL 61938  
**1-800-573-1940**

SBL Effingham Laboratory  
905 North Maple • Effingham, IL 62401  
**1-877-796-LABS (5227)**

[testmenu.com/sarahbush](http://testmenu.com/sarahbush)