

St. Joseph's/Candler Health System

StatStrip Glucose Meter Training Quiz

Please circle the correct answer

1. The purpose of the docking station is:
 - A. Charge the battery in the meter
 - B. Charge the spare battery
 - C. Connect the meter to a data management system
 - D. All of the above
2. What is the expiration date of the test strips after they have been opened?
 - A. The expiration date does not change
 - B. 6 months or the printed expiration date on the vial, whichever comes first
 - C. 14 days
3. What is the expiration date of the quality control material once opened?
 - A. The expiration date does not change
 - B. 1 month
 - C. 3 months or the printed expiration date on the vial, whichever comes first.
4. The meter should always be held level or pointed slightly downwards when applying the sample to the strip to keep liquid from running down into the meter and damaging it.
 - A. True
 - B. False
5. The sample must be applied to the end of the strip and must fill the strip completely upon touching the blood drop. If it does not fill completely, you may touch the strip to the blood drop a second time until the countdown begins.
 - A. True
 - B. False
6. If data transfer was incomplete before the meter was removed from the docking station, it is still OK to use the meter.
 - A. True
 - B. False

Operator Signature: _____ Date: _____