

St. Joseph's/Candler Health System

Clinitek Urine Pregnancy Training Quiz

Employee Name (Please Print) _____ Signature _____

Operator ID _____ Department _____ Date _____

General Urine Pregnancy Testing Questions

1. To enter patient identification, the operator must use:
 - A. The current date
 - B. The operators ID
 - C. Full name and Date of Birth
 - D. Date of Birth
 - E. All of the above.
2. True or False – hCG can be detected as early as 14 days after conception.
3. True or False – The Clinitek requires maintenance and QC every day.
4. What is recommended for weekly maintenance
 - A. Dust the outside of the analyzer and wipe the result window with a mild detergent.
 - B. There is no required weekly maintenance
 - C. Clean the Calibration Bar with a mild detergent
 - D. Clean the Test Table and the Test Table insert
5. With a borderline result:
 - A. Recommendation is to recollect and retest in 48-72 hours.
 - B. Result falls in the reference region with hCG concentration less than 25mIU/mL.
 - C. Can be manually resulted as negative.
 - D. A & B.
6. Once the operator selects "START":
 - A. The operator must push the test table into the analyzer.
 - B. The operator is allowed to manually read and enter results.
 - C. "Analyzing" will be displayed until testing is complete and the test table will return.
 - D. None of the above
7. True or False – If stored correctly, urine samples are stable for testing up to 72 hours.
8. For valid pregnancy results:
 - A. The analyzer will detect a colored line in the control region and the reference region.
 - B. Positive results will detect a colored line in the control region and the test region.
 - C. Negative results will detect a colored line in the control region, the reference region, not the test region.
 - D. All of the above.

***To be completed by trainer/POC Representative**

Score ____ of ____ / ____% POC Representative _____