

St. Joseph's/Candler Point of Care Department **Coag-Sense PT/INR Test Strip Shipment QC Log Sheet**

Location: _____

Frequency of QC Testing = Each New Shipment / New Lot #

Date Received	Date Tested	Tested by	PT Test Strips Lot #	<u>Low Control</u>					<u>High Control</u>				
				Lot #	Exp. Date	Expected Range (sec.) *	PT Sec.	Pass (v)	Lot #	Exp. Date	Expected Range (sec.) *	PT Sec.	Pass (v)

*Expected range for the low and high control strips lot can be found on the test strip kit box.

Note: Control strip lot numbers will be different than patient test strip and strip kit lot numbers. Run one high and one low control strip with each new shipment of strips received and record the results. If your shipment includes multiple lot numbers, run controls from each lot received. Please run controls when you receive your shipment as controls have a limited shelf life. Write **"QC DONE"** on the top of each box in your shipment as they will all be qualified at that point. Refer to your user manual for additional information. **Call CoaguSense technical support at 1-866-903-0890 if you have any questions.**