

St. Joseph's/Candler Point of Care Department
Coag-Sense Training Assessment Quiz

Mark True (T) or False (F)

1. ___ One can change the Date/Time and Format by pressing Settings/Device Settings
2. ___ When using a Control or Patient Test Strip if the Strip Lot Confirmation screen does not match the Lot # (six-digit number) and Barcode # then NFC Tag should be scanned prior to testing
3. ___ The Operating Temperature for the Meter is 32°F to 122°F (0°C to 50°C)
4. ___ Both Control Strips use the same Control Activation Solution
5. ___ A Strip should be used within 10 min after its removal from the pouch
6. ___ Any Standard User can create or delete Users
7. ___ QC Lockout feature can be overridden by the Admin
8. ___ Patient ID information can be scanned or manually entered in the field prior to testing
9. ___ After strip insertion the meter stays for 50 seconds in the "Please Wait Until Warmup Is Complete" phase
10. ___ After the fingerstick is done, the sample should be drawn in Sample Transfer Tube within 15 seconds
11. ___ After filling the Sample Transfer Tube, the sample can be applied to green flashing light of the strip, within one minute
12. ___ The meter can store 2000 patient test results and 500 control results in its Memory
13. ___ To access the Memory, one has to press the Control Button
14. ___ User may export recorded results into the electronic medical record system or print individual results via Printer
15. ___ After the test results are reported, if they are inconsistent with expected results the user should re-test per instructions and then contact their doctor
16. ___ The meter can be cleaned and disinfected by immersing in a cleaning solution
17. ___ If there any questions about the system call Technical Support at 1-866-903-0890
18. ___ The meter can be powered ON/OFF by pressing and holding the power button.

Quiz results: number of correct answers _____

Trainee Name _____ Date: _____

Training Site: _____

Trainer Name: _____

Trainer signature: _____ Date: _____