

St. Joseph's/Candler Health System

Clinitek Urinalysis Training Quiz

Employee Name (Please Print) _____ Signature _____

Operator ID _____ Department _____ Date _____

General Urinalysis Questions

1. To enter patient identification, the operator must use:
 - A. The current date
 - B. The operators ID
 - C. Full name and Date of Birth
 - D. Date of Birth
 - E. All of the above.
2. Number the following steps (1-5) to correspond with the correct order in which they should be performed.

_____ Place the test strip onto the test table
_____ Scan or enter the Operator ID and Patient ID
_____ Enter the test strip lot number
_____ Select strip test and ENTER
_____ Review results
3. True or False – The Clinitek calibrates automatically before each test measurement by reading the white calibration bar.
4. True or False – The Clinitek requires maintenance and QC every day.
5. What is recommended for weekly maintenance
 - A. Dust the outside of the analyzer and wipe the result window with a mild detergent.
 - B. There is no required weekly maintenance
 - C. Clean the Calibration Bar with a mild detergent
 - D. Clean the Test Table and the Test Table insert
6. Once the operator selects "START":
 - A. The operator must push the test table into the analyzer.
 - B. The operator is prompted to select the appropriate color and clarity.
 - C. "Analyzing" will be displayed until testing is complete and the test table will return.
 - D. B and C
7. True or False – If stored correctly, Mutlistix are stable until the expiration date on the bottle.
8. After running liquid control samples, to verify correct results:
 - A. Results will reflect those of a normal patient
 - B. The results will be the exact same as the previous QC result
 - C. The analyzer will display "PASS"
 - D. The results will reflect manufacturer's guidelines.

***To be completed by trainer/POC Representative**

Score _____ of _____ / _____ % POC Representative _____