

St. Joseph's/Candler Health System

Strep A Test Training Quiz

Employee Name (Please Print) _____ Signature _____

Operator ID _____ Department _____ Date _____

General Group A Strep Testing Questions

1. Test kits are stored at what temperatures? _____ -- _____

2. Test strips are stable:

- A. At Room Temperature
- B. Until the printed expiration date on the box
- C. If left in original pouch unopened
- D. With passing QC after 30 days of storage
- E. All of the above are true

3. Number the following steps (1-5) to correspond with the correct order in which they should be performed.

_____ Set a timer for five minutes

_____ With arrows pointing down, place the test strip in the solution

_____ Mix the swab 10 times in the solution

_____ Review and record results

_____ Squeeze excess reagent as you remove the swab so that most liquid stays in the tube

_____ Drop Reagent 1 (A) into the test tube, then drop Reagent 2 (B) into the test tube

4. How many drops of Reagent 1 (A) and 2 (B) are used?

- A. 2
- B. 3
- C. It doesn't matter as long as they are both used
- D. None of the above

5. Enter Determined Results:

A. If no bands appear in the test window, the results are: _____

B. If two bands appear in the test window, the results are: _____

C. If one band appears in the test window, the results are: _____

***To be completed by trainer/POC Representative**

POC Representative _____

Score ____ of ____ / ____%