

Order of Draw		
Tube Color	Tube Name	Mix by Inversion
 <p>Aerobic (Blue Label) Anaerobic (Pink Label) Pediatric (Yellow Label)</p>	Blood Cultures	8-10 times (collect aerobic bottle first)
	Light Blue (Sodium Citrate) Tubes	4 times
	Red or Gold (Serum) Tubes	5-10 times (Clotting time: 30 minutes)
	Dark Green or Light Green (Heparin) Tubes	8-10 times
	Lavender or Pink (EDTA) Tubes	8-10 times
	Grey (Sodium Fluoride) Tube	8-10 times