


# BD Cup Kit Instructions

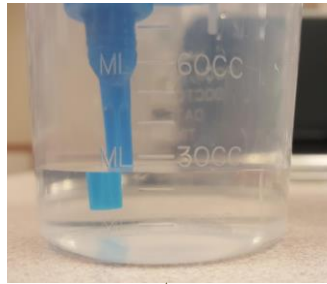

Example of 25 mL

**Please Note:** When filling tubes via the transfer device, ensure you are transferring on a flat surface and hold the tubes for the entire transfer. Also ensure the tubes have filled *completely* before taking them off the transfer device.


Acceptable Sample Types:  
Clean Catch  
*Transfer immediately after collection*

Is there  $\geq 25$  mL of urine?

Yes

First: fill C&S (Grey) tube,  
Second: fill UAP (Cheery Red Tiger) tube,  
Third: fill No-Additive (Red) tube

Mix all 3 tubes by inverting  
(Refrigerate No-Additive tube or transport immediately)

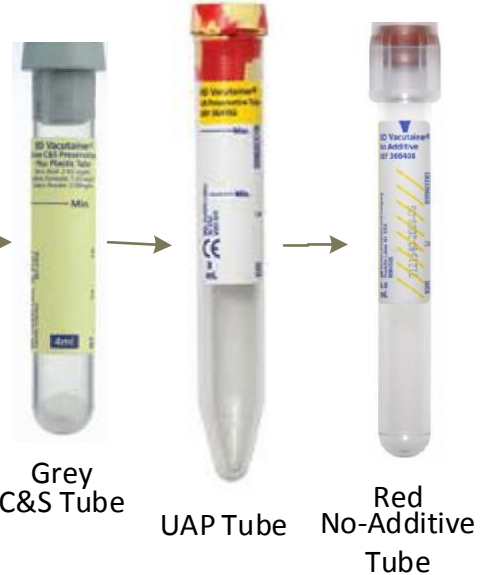

Grey C&S Tube

UAP Tube

Red No-Additive Tube

No

Use straw to transfer urine  
Anchor straw against the side of the cup by using your index finger and thumb (as pictured)



Assess volume of urine, if  $< 10$  mL of urine, fill only the No-Additive (Red) tube

Mix tubes by inverting  
(Refrigerate No-Additive tube or transport immediately)



Red No-Additive Tube