



Collection and Transport of Non Blood Specimens

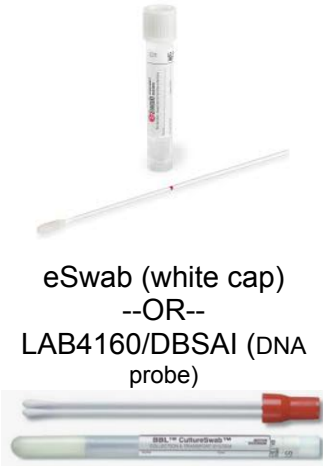
For up-to-date information see TriHealth Test Menu: [TriHealth Test Directory | Home \(testmenu.com\)](https://testmenu.com)

Bacterial Cultures LAB7172/WDC LAB3054/ANER LAB4171/URESC LAB3334/GENC LAB3293/EYEC LAB3271/EARC LAB3329/FUNS  eSwab (white cap) *Provider collected*	Viral Culture/PCR LAB4275/CORON LAB4300/4PLEX LAB4128/HSVC (culture, no subtype) LAB4150/HSVCT (culture, w/ subtype) LAB4175/HSVLT (DNA swab, w/ subtype) LAB4131/VRCNR (non-resp culture) LAB4129/VRESP (respiratory culture)  Send to lab ON ICE! Remel M4RT (or UTM)	Mycoplasma/ Ureaplasma LAB3530/UPLAB  Send to lab ON ICE! ARUP Transport Media	Urine Culture URNC/LAB3738  Urine C&S Vacutainer
GC/Chlamydia & Trich Endocervical/Urethral LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only) LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium) *For patients under 14 years old order: LAB7558/CHTMA (GC & CH)  Aptima Unisex Swab *Provider collected*	GC/Chlamydia & Trich Vaginal/Rectal/Throat LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only) LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium) *For patients under 14 years old order: LAB7558/CHTMA (GC & CH)  Aptima Multitest Swab *Self-collect allowed*	GC/Chlamydia & Trich Urine LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only) LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium) *For patients under 14 years old order: LAB7558/CHTMA (GC & CH)  Aptima Urine Probe	Pertussis PCR LAB590/BPER1  eSwab (blue cap) Vaginitis Panel NAAT LAB13561/BVP BV, Candida, Trich  Do not use lubricant. *Self-collect allowed*



Collection and Transport of Non Blood Specimens

For up-to-date information see TriHealth Test Menu: [TriHealth Test Menu: TriHealth Test Directory | Home \(testmenu.com\)](https://www.trihealth.com/testmenu)

Strep A Backup testing: (Throat swabs only) LAB4171/URESC (culture) LAB7907/GRASP (PCR)  eSwab (white cap) --OR-- LAB4160/DBSAI (DNA probe) BBL Culture Swab with Stuart or Amies media (no Cardinal swabs)	C. Difficile LAB8308/CDIFM* *formed stool will be rejected Rotavirus ROTVI/LAB3635 Helicobacter pylori HPYAG/LAB3829 Fecal Lactoferrin STWBC/LAB3781 Entamoeba histolytica ENHIS/LAB7500 Calprotectin fecal CALPF/LAB7292 Fat, fecal qual FTOTQ /LAB3303 Pancreatic elastase PANEF/LAB7469 Stool Testing Send to lab ON ICE!  Clean container, no preservative	
 Cary-Blair keep Room Temp Stool Culture Salmonella, Shigella, Campylobacter, E coli, Yersinia, Plesiomonas, Aeromonas, Vibrio STLC/LAB3668 E. coli Shiga-like Toxin ECTOX/LAB7246	 Formalin keep Room Temp Giardia and Cryptosporidium Antigen OAPA/LAB 7524 Microsporidium OAPMS/LAB3546 Cyclospora/Isospora OAPCI/LAB3241	 Total Fix, keep Room Temp Complete Ova & Parasite exam for patient with travel history OAPTI/LAB3517 Clean container, no preservative (above) Refrigerated -OR- Hemoccult SENA cards only (BLUE stripe) Room temp  Hemoccult HMCLT/LAB3362

