










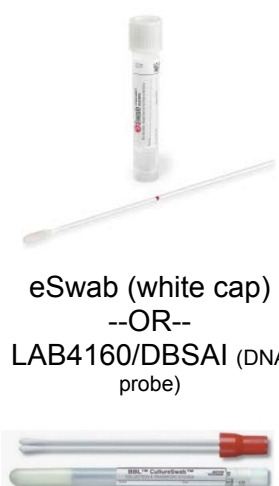




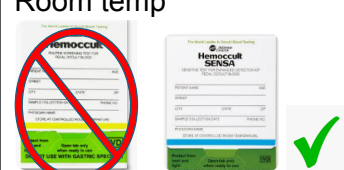
Collection and Transport of Non Blood Specimens

For up-to-date information see TriHealth Test Menu: [TriHealth Test Directory | Home \(testmenu.com\)](http://TriHealth Test Directory | Home (testmenu.com))

<p>Bacterial Cultures</p> <p>LAB7172/WDC LAB3054/ANER LAB4171/URESC LAB3334/GENC LAB3293/EYEC LAB3271/EARC LAB3329/FUNS</p>  <p>eSwab (white cap) *Provider collected*</p>	<p>Viral Culture/PCR</p> <p>LAB4275/CORON LAB4300/4PLEX</p> <p>LAB4128/HSVC (culture, no subtype) LAB4150/HSVCT (culture, w/ subtype) LAB4175/HSVL (DNA swab, w/ subtype) LAB4131/VRCNR (non-resp culture) LAB4129/VRESP (respiratory culture)</p>  <p style="text-align: center;">Send to lab ON ICE!</p> <p>Remel M4RT (or UTM)</p>	<p>Mycoplasma/ Ureaplasma</p> <p>LAB3530/UPLAB</p>  <p style="text-align: center;">Send to lab ON ICE!</p> <p>ARUP Transport Media</p>	<p>Urine Culture URNC/LAB3738</p>  <p>Urine C&S Vacutainer</p>
<p>GC/Chlamydia & Trich Endocervical/Ureathral</p> <p>LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only)</p> <p>LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium)</p> <p><small>*For patients under 14 years old order: LAB7558/CHTMA (GC & CH)</small></p>  <p>Aptima Unisex Swab *Provider collected*</p>	<p>GC/Chlamydia & Trich Vaginal/Rectal/Throat</p> <p>LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only)</p> <p>LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium)</p> <p><small>*For patients under 14 years old order: LAB7558/CHTMA (GC & CH)</small></p>  <p>Aptima Multitest Swab *Self-collect allowed*</p>	<p>GC/Chlamydia & Trich Urine</p> <p>LAB7336/APTCG (GC & CH) LAB7338/APTGC (GC only) LAB7337/APTCH (CH only)</p> <p>LAB4201/TRTMA (Trich) LAB543/MGENI (M.genitalium)</p> <p><small>*For patients under 14 years old order: LAB7558/CHTMA (GC & CH)</small></p>  <p>Aptima Urine Probe</p>	<p>Pertussis PCR LAB590/BPER1</p>  <p>eSwab (blue cap)</p>
<p>Vaginitis Panel NAAT LAB13561/BVP BV, Candida, Trich</p>  <p>Do not use lubricant. *Self-collect allowed*</p>			

Collection and Transport of Non Blood Specimens

For up-to-date information see TriHealth Test Menu: [TriHealth Test Directory | Home \(testmenu.com\)](http://TriHealth Test Directory | Home (testmenu.com))

<p>Strep A Backup testing: (Throat swabs only) LAB4171/URESC (culture) LAB7907/GRASP (PCR)</p>  <p>eSwab (white cap) --OR-- LAB4160/DBSAI (DNA probe)</p> <p>BBL Culture Swab with Stuart or Amies media (no Cardinal swabs)</p>	<p>C. Difficile LAB8308/CDIFM* *formed stool will be rejected</p> <p>Rotavirus ROTVI/LAB3635</p> <p>Helicobacter pylori HPYAG/LAB3829</p> <p>Fecal Lactoferrin STWBC/LAB3781</p> <p>Entamoeba histolytica ENHIS/LAB7500</p> <p>Calprotectin fecal CALPF/LAB7292</p> <p>Fat, fecal qual FTOTQ /LAB3303</p> <p>Pancreatic elastase PANEF/LAB7469</p> <div style="text-align: center;"> <p>Stool Testing</p>  <p>Clean container, no preservative</p> </div>	
 <p>Cary-Blair keep Room Temp</p> <p>Stool Culture Salmonella, Shigella, Campylobacter, E coli, Yersinia, Plesiomonas, Aeromonas, Vibrio STLC/LAB3668</p> <p>E. coli Shiga-like Toxin ECTOX/LAB7246</p>	 <p>Formalin keep Room Temp</p> <p>Giardia and Cryptosporidium Antigen OAPA/LAB 7524</p> <p>Microsporidium OAPMS/LAB3546</p> <p>Cyclospora/Isospora OAPCI/LAB3241</p>	 <p>Total Fix, keep Room Temp</p> <p>Complete Ova & Parasite exam for patient with travel history</p> <p>OAPTI/LAB3517</p> <div style="text-align: right;"> <p>Clean container, no preservative (above) Refrigerated -OR- Hemocult SENSEA cards only (BLUE stripe) Room temp</p>  <p>Hemocult</p> <p>HMCLT/LAB3362</p> </div>

Q] : [ç^á/æ á/æ ; ; ^) dŮ-^ &ç^Á çæç * Á FCBEG ĚŮŮĜ GĚ^! • ā } Á Ě DĚ [] ÁŮ [[áŮ] ^ &ā ^ } Á / : æ } [[ŮŮ [] çæ ^! • Ě Ě ^! • ā * BŮ d æā } c
A' æ [āā [* ^ Á ^ æĀ Á æ^

Ô [] d [[[^ á / æ] ^ ÁŮ Á Ĩ ĩ e ĩ ĚŮ ĩ ç á Ā } Á Ě Ĩ B E G Á Ĩ K E Ě Ů Ů Ě Ě ^ Á Ĥ Á Ĩ