




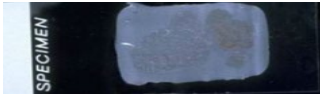


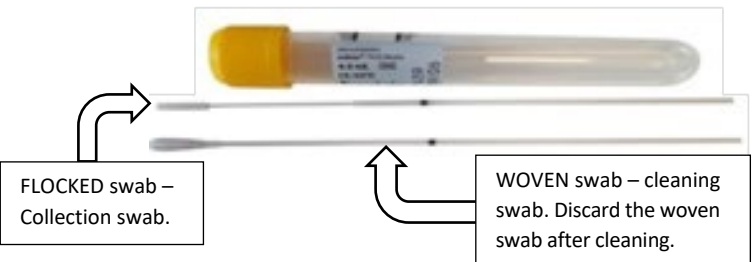



TO ENSURE ACCURACY OF MOST UP TO DATE INFORMATION, PLEASE DO NOT PRINT & DISTRIBUTE

UCDH MOLECULAR ACCEPTABLE SPECIMEN COLLECTION CONTAINERS

TEST	SPECIMEN COLLECTION CONTAINER	
<ul style="list-style-type: none"> HER-2 FISH (Breast & Gastric) 	FFPE Only	
<ul style="list-style-type: none"> Factor V Leiden Mutation Prothrombin (Factor II) DNA Mutation 	K2 EDTA	
<ul style="list-style-type: none"> Hereditary Hemochromatosis (HFE) Cystic Fibrosis DNA Mutations 	EDTA	
<p>Viral Loads (Quantitative PCR)</p> <ul style="list-style-type: none"> HIV-1 CMV HBV HCV EBV BK -Blood 	Plasma from EDTA	
<ul style="list-style-type: none"> BK Viral Load – Urine 	Cobas PCR Media – Urine Collection Kit	
<ul style="list-style-type: none"> Chlamydia Trachomatis Neisseria Gonorrhoeae (CT/NG) – <p>URINE</p>	Cobas PCR Media – Urine Collection Kit	



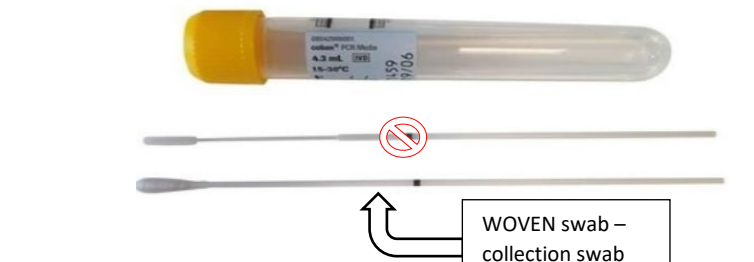

TO ENSURE ACCURACY OF MOST UP TO DATE INFORMATION, PLEASE DO NOT PRINT & DISTRIBUTE

UCDH MOLECULAR ACCEPTABLE SPECIMEN COLLECTION CONTAINERS

TEST	SPECIMEN COLLECTION CONTAINER	
<ul style="list-style-type: none"> Chlamydia Trachomatis / Neisseria Gonorrhoeae (CT/NG) <p>FEMALE GENITAL – ENDOCERVICAL COLLECTION</p>	<p>Cobas PCR Media Dual Swab Kit ONLY (refer to kit instructions)</p>	 <p>FLOCKED swab – Collection swab.</p> <p>WOVEN swab – cleaning swab. Discard the woven swab after cleaning.</p>
<ul style="list-style-type: none"> Chlamydia Trachomatis / Neisseria Gonorrhoeae (CT/NG) <p>FEMALE GENITAL – VAGINAL COLLECTION (Clinician and Self-Collections)</p>	<p>Cobas PCR Media Uni or Dual Swab Kit (Woven swab ONLY)</p>	 <p>WOVEN swab – collection swab</p>
<ul style="list-style-type: none"> Chlamydia Trachomatis / Neisseria Gonorrhoeae (CT/NG) <p>ANORECTAL COLLECTION (Clinician and Self-Collections)</p>	<p>Cobas PCR Media Uni or Dual Swab Kit (Woven swab ONLY)</p>	 <p>WOVEN swab – collection swab</p>
<ul style="list-style-type: none"> Chlamydia Trachomatis / Neisseria Gonorrhoeae (CT/NG) <p>THROAT COLLECTION (Clinician and Self-Collections)</p>	<p>Cobas PCR Media Uni or Dual Swab Kit (Woven swab ONLY)</p>	 <p>WOVEN swab – collection swab</p>







TO ENSURE ACCURACY OF MOST UP TO DATE INFORMATION, PLEASE DO NOT PRINT & DISTRIBUTE

UCDH MOLECULAR ACCEPTABLE SPECIMEN COLLECTION CONTAINERS

TEST	SPECIMEN COLLECTION CONTAINER	
<ul style="list-style-type: none"> Human Papillomavirus (HPV) DNA <p>CERVICAL COLLECTION (Clinician Collections)</p>	<p>Hologic ThinPrep PAP Test Vial</p> <p><i>Cervical collections using PAP collection devices (spatula brush, or broom) into Hologic ThinPrep PAP Test vial. Refer to Lab Test Directory for detailed instructions.</i></p> <p><i>Note: ThinPrep PreservCyt Solution (NON-GYN) vial is unacceptable for HPV DNA testing.</i></p>	 
<ul style="list-style-type: none"> Human Papillomavirus (HPV) DNA <p>VAGINAL COLLECTION (Patient Self-Collections only)</p> <p><i>Note: Clinician-collected vaginal specimens using Copan FLOQSwabs are unacceptable and will be rejected.</i></p> <p><i>Note: Clinician-collected vaginal specimens using PAP collection devices (spatula, brush, or broom) will be forwarded to send out reference lab when necessary.</i></p>	<p>Hologic ThinPrep PAP Test Vial</p> <p><i>Vaginal self-collections using Copan FLOQSwab 552C into Hologic ThinPrep PAP Test vial. Refer to Lab Test Directory for detailed instructions.</i></p> <p><i>Note: ThinPrep PreservCyt Solution (NON-GYN) vial is unacceptable for HPV DNA testing.</i></p>	<p>Copan FLOQSwab 552C – Lawson Order #198678</p>  

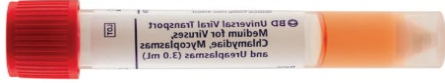


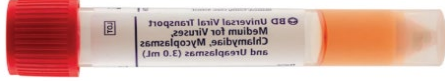
TO ENSURE ACCURACY OF MOST UP TO DATE INFORMATION, PLEASE DO NOT PRINT & DISTRIBUTE

UCDH MOLECULAR ACCEPTABLE SPECIMEN COLLECTION CONTAINERS

TEST	SPECIMEN COLLECTION CONTAINER	
<ul style="list-style-type: none"> Respiratory Viral Panel 	BD UTM	
	M4	
	M4 RT	
<ul style="list-style-type: none"> Bordetella Pertussis/Parapertussis DNA 	BD UTM	
	M4	
	M4 RT	

TO ENSURE ACCURACY OF MOST UP TO DATE INFORMATION, PLEASE DO NOT PRINT & DISTRIBUTE

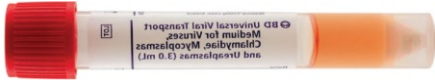UCDH MOLECULAR ACCEPTABLE SPECIMEN COLLECTION CONTAINERS

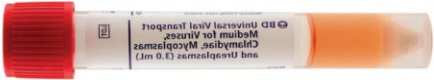TEST	SPECIMEN COLLECTION CONTAINER	
<ul style="list-style-type: none"> Covid-2019 RNA Qual 	BD UTM	
	M4 RT	 
<ul style="list-style-type: none"> Covid-2019 + Flu A/B 	BD UTM	
<p align="center"> For additional details regarding test requisitioning, specimen collection and transport requirements, results, and interpretation, search individual tests in our Lab Test Directory at https://www.testmenu.com/ucdavis Any further questions should be routed to Client Services Department: Phone: (916) 734-7373 Fax: (916) 737-1001 </p>		