



**Copan UTM-RT Viral Transport Medium**

<b>Use for</b>	<i>Epic order displays as:</i> <ul style="list-style-type: none"> <li>Influenza A/B &amp; SARS-CoV-2 PCR Combo for Rapid Response Lab (FLUCO)</li> </ul>
<b>Supplied by</b>	<ol style="list-style-type: none"> <li>Store House</li> <li>House Supervisor (afterhours)</li> </ol>
<b>Procedure for collection</b>	<ul style="list-style-type: none"> <li>ONLY USE COPAN UTM-RT!</li> <li>Use the same swab to collect left and right anterior nares, nasal mid-turbinate, or nasopharyngeal sites</li> <li>Place swab in liquid and snap at score mark</li> <li>Close cap tightly. Make sure cap is seated properly, leaking specimens will be rejected!</li> <li>Place specimen into biohazard bag</li> </ul>
<b>Special Considerations</b>	<ul style="list-style-type: none"> <li>Test includes Influenza A/B and COVID-19 PCR tests. No additional backup tests are needed.</li> </ul>


**FOR COVID SPECIMENS:**

- Walk specimen down to lab
- Announce delivery of COVID specimen to lab staff member
- DO NOT send specimen via pneumatic tube system



**iClean Flocked Swab**

<b>Special Considerations</b>	<b>USE FOR ALL COVID-19 COLLECTIONS, INCLUDING RAPID AND ROUTINE PCR</b>
-------------------------------	--

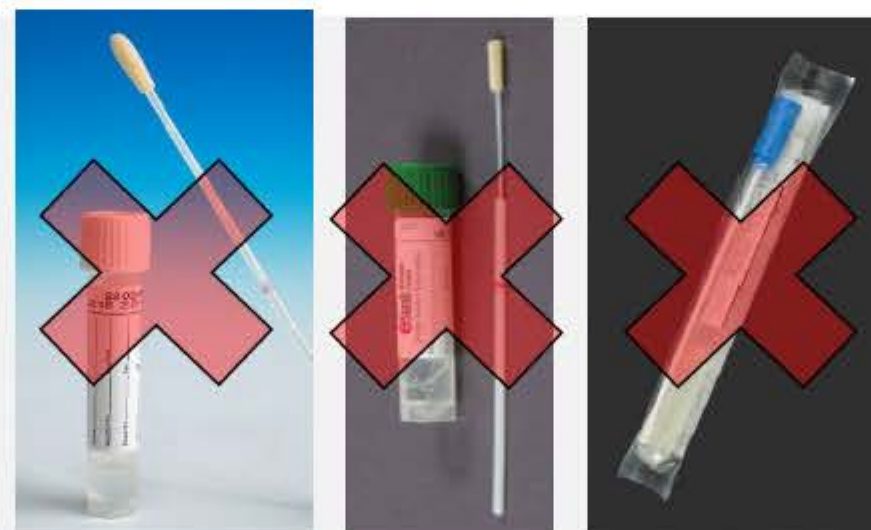

**COPAN eSWAB, BBL CultureSwab  
DO NOT USE FOR COVID-19 TESTING**

<b>Special Considerations</b>	<b>DO NOT USE FOR COVID-19 TESTING</b>
-------------------------------	--



**Viral Transport Medium (VTM) with Red or White Cap**

<b>Use for</b>	<i>Epic order displayed as:</i> <ul style="list-style-type: none"> <li>COVID-19 Coronavirus Detection Assay at UCSD Labs</li> </ul>
<b>Supplied by</b>	<ol style="list-style-type: none"> <li>Store House (OK to store limited supply on the unit at room temp until use)</li> <li>House Supervisor (afterhours)</li> </ol>
<b>Procedure for collection</b>	<ul style="list-style-type: none"> <li>Use the same swab to collect left and right anterior nares, nasal mid-turbinate, or nasopharyngeal sites</li> <li>Can also accept oropharyngeal swabs, tracheal aspirate, or bronchial washings</li> <li>DO NOT USE THICK RAYON TIPPED SWABS FOR NASOPHARYNGEAL COLLECTION ONLY FLOCKED SWABS</li> <li>Place swab in liquid and snap at score mark</li> <li>Close cap tightly. Make sure cap is seated properly, leaking specimens will be rejected!</li> </ul>
<b>Special Considerations</b>	<ul style="list-style-type: none"> <li>Specimens are good for 72 hours after collection at room temperature</li> <li><i>Can also be used for:</i> <ul style="list-style-type: none"> <li>Upper Respiratory Pathogen Nucleic Acid Test (nasopharyngeal only)</li> <li>Influenza/RSV PCR testing (nasopharyngeal only)</li> <li>HSV 1/2 and VZV PCR Lesion</li> <li>Send-out viral culture</li> </ul> </li> </ul>